

Technical Data Sheet

Type II Face Mask Blue/White B442561

Product Description

Type II Non-woven 3 Ply Disposable Face Mask in Blue/White with ear loop
EN 14683
50 Masks per box



Non-Woven
Medical Devices Class 1 Non-Sterile

Product Features

- Spunbond+Meltblown+Spunbond
- Non-woven 3 Ply design
- Disposable, single use, breathable light face mask
- 3-Fold Face Mask
- Soft adjustable nose pad ensures a comfortable fit
- Elasticated ear loop
- Latex Free
- Size 17.5 x 9.5cm
- EN 14683
- Type II
- Box 50pcs



Intended Use

For use in general environments or public environments

Storage

The items should be stored in a cool dry place with ventilation

Shelf Life

5 years

Instructions For Use

1. Before putting on mask clean hands with Alcohol based sanitizer or soap and water
2. Cover your mouth and nose with the mask and make sure to have a minimum gap between your face and the mask
3. Press the nose strip gently to fit the nose and surrounding face,
4. Avoid touching the mask while using it
5. Replace mask with a new one when the mask is damp
6. Dispose the mask after use